medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377
USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
Tel: +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medidanmark.dk

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32-11 24 25 60
F +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

4 049772 365630

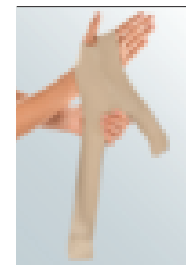
E012036 / 11.2016

medi

CIRCAID

circaid® juxtafit® essentials

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circaid® juxtafit®
hand wrap

circaid® juxtafit®
essentials glove

circaid® juxtafit®
essentials glove with
dorsum strap

medi. I feel better.

circaid® juxtafit® essentials hand wrap

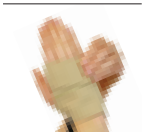
Step 1

Step 2

Step 3

Step 4

Step 5

Step 6

circaid® juxtafit® essentials glove

Step 1

Step 2

Step 3

Step 4

circaid® juxtafit® essentials golve with dorsum strap

Step 1

Step 2

Step 3

Step 4

circaid® juxtafit® essentials glove removal strap

Step 1

Step 2

Step 3

circaid® juxtafit® essentials

Purpose

This garment is a single patient, reusable, non-invasive, non-sterile device designed to provide compression to the arm of patients with lymphatic disorders. The armsleeve consists of a series of juxtapositioned inelastic bands extending from the central portion of the wrap. It is designed to cover the arm from the wrist to the axilla.

Additionally there is a hand wrap included, which provides compression to the hand, from the wrist to the base of the fingers.

Indications

- Lymphedema

Relative contraindication

Due to a known contraindication the attending doctor weighs up

the wearing of a medical compression garment (for a suitable indication such as Lymphedema, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the arm
- Allergy to compression materials

Absolute contraindication

A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. Lymphedema, etc.).

- Severe peripheral arterial disease
- Decompensated congestive heart failure
- Septic phlebitis
- Phlegmasia cerula dolens
- Moderate peripheral arterial disease
- Acute Infection

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Italia S.r.l.
Via Guido Rossa 28
40033 Casalecchio di Reno-Bologna
Italy
T +39 051 613 24 84
F +39 051 613 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T +47 52 84 45 00
F +47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Orient M.E.A. FZC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
T +971 4 2557344
F +971 4 2557399
md@taurusgroup.net

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